

【INSTRUCTION FOR USE】 GUIDEWIRE

SCOPE OF APPLICATION

Guidewire(Stainless steel guidewire&Nitinol guidewire) is applicable as an invasive medical devices. It is intended for guiding and assisting insertion of percutaneous catheter.

CONTRA-INDICATIONS

None known.

THE USE OF THE ENVIRONMENT AND STORAGE

This product should be stored in a cool dry environment, avoid direct sunlight.

PLEASE PAY ATTENTION TO THE FOLLOWING POINTS

- 1,It must be operated by trained doctor.
- 2,This product is sterilized by ethylene oxide, if damaged packaging use is strictly prohibited.
- 3,This product is for one-time use, do not repeat sterilization and repeat use.
- 4,This product should not be used in the process of arbitrarily change its structure.

THE STRUCTURE AND SPECIFICATIONS

The guidewire(Stainless steel guidewire&Nitinol guidewire) is composed of core wire, and spring coil.

Materials: core wire: stainless steel or nickel titanium alloy;
the spring coil: stainless steel.

Specifications: length: 20cm ~ 260cm; diameter (Inch):0.012,0.014,0.018,0.021,0.025,0.028, 0.032, 0.035, 0.038; the shape of tip: J tip, straight tip.

INSTRUCTIONS FOR USE

- 1, Carefully pull the guide wire insertion of the catheter and guide wire tip propulsion, taking care not to damage the wire tip. You may need to use the wire guide advancer during the process of insertion.
- 2, The torque device on the guide wire, and determining the position in accordance with the needs of. Screw the lid fixed it well.
- 3, Under fluoroscopy and carefully push the rotating filament guide, to select suitable vessels. (warning: make using X-ray fluorescence to observe the whole mobile guide wire in the blood vessels. The corresponding mobile observation of distal end of the guide wire The situation is rear can move or twist wire; otherwise, the possible guide wire damage, such as tip separation And / or vascular injury. Always be careful slow push in or withdraw from the guide wire.)
- 4, The catheter to advance to the selected vessel position, keeping the guide wire is fixed.

WARNING

- 1 the duration of the sterilization of products for three years, please confirm the sterilization date before use, prohibited the use if expired.
- 2 this product is a disposable product, it is strictly prohibited to repeated sterilization, repeated use.

SHUNMEI MEDICAL CO.,LTD.

Add: R401 of building B, No.8 of 1st Jinglong Road, Baolong Industrial Zone, LongGang District 518116 Shenzhen, Guangdong , China.

Tel : 0752-3306949

Email : nora@shunmed.com

Web : www.shunmed.com

Lotus NL B.V.

Koningin Julianaplein 10, 1e Verd,
2595AA, The Hague, Netherlands.
Tel: +31644168999

Email: peter@lotusnl.com



Do not reuse

Sterilization using ethylene oxide



Attention, see instruction for use



Fragile



Keep away from sunlight



Consult instructions for use



Do not resterilize



Store in cool place



Keep dry



damage to packing do not use



Use by

Lot Number

Catalog Number